

# Mini Pillow



# Mini Pillow

Mini Pillow is a hand-operated irrigation device that is designed to enhance visibility by efficiently clearing debris during ureterorenoscopy.

## Controlled Irrigation


The In-Line Flow Switch creates a comfortable single-handed on/off control of the flow and prevents inadvertent spillage of irrigation fluid.

## Dual Flow

- A continuous low-intensity background flow regulated by adjusting the height of the irrigation fluid reservoir above the patient's level and;
- An accelerated, on demand flow achieved with compression of the pillow.

## Tubing Extension


- The Mini Pillow system includes a tubing extension. This can be attached between the In-Line Flow Switch of the Mini Pillow unit and endoscope.
- The extension allows the assistant to control the irrigation in a comfortable position.

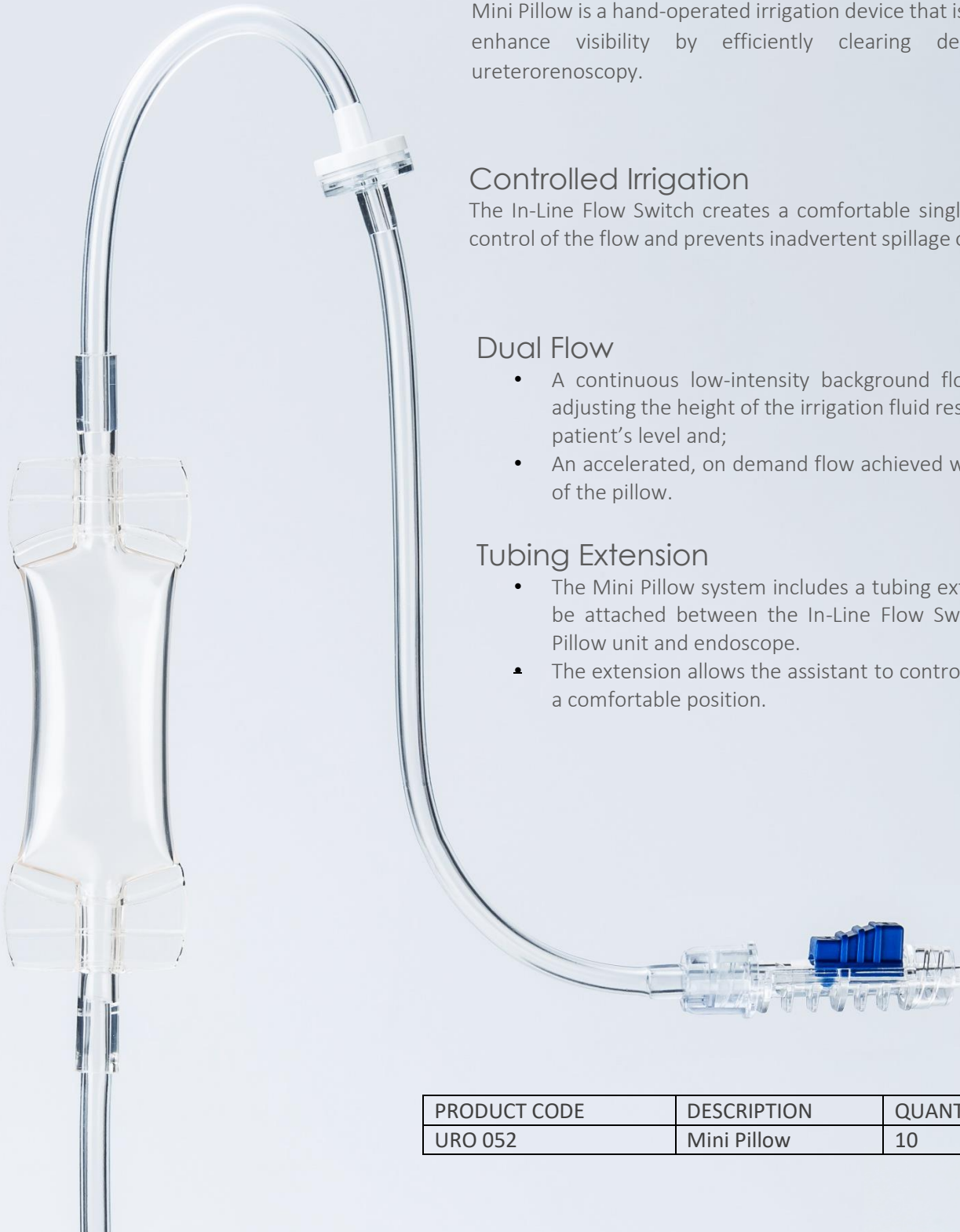

| PRODUCT CODE | DESCRIPTION | QUANTITY |
|--------------|-------------|----------|
| URO 052      | Mini Pillow | 10       |



Wismed CC  
71 Princess Alice Avenue,  
Durban, 4001, South Africa  
Tel: +27 31 206 0177  
Email: [orders@wismed.co.za](mailto:orders@wismed.co.za)  
[www.wismed.co.za](http://www.wismed.co.za)

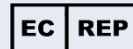

TNMC Devices (Llc)  
Ulica Franje Krezme 1  
10000 Zagreb, Croatia  
Tel: +38 59 1622 9658  
Email: [tnmchr@tnmc.biz](mailto:tnmchr@tnmc.biz)

